



RECEIVED

JUN 17 2015

10111 WEST JEFFERSON BOULEVARD, CULVER CITY, CALIFORNIA 90232
TELEPHONE 310-837-6500 FAX 310-837-9563 www.moldex.com

REVISION 1

Turner Construction Co.

April 14, 2015

Subject: NIOSH APPROVED MOLDEX DISPOSABLE RESPIRATORS WITH EXHALE VALVE

Re: Moldex User Voluntary Recall/Stop Use/Relabel Shelf Life Notice

Series: 2300 N95, 2700 N95, EZ23N95 and EZ23S N95

Model Numbers: 2236, 2300N95, 2300V, 2301N95, 2307N95, 2350N95, 2357N95, 2700N95, 2700V, 2701N95, 2707N95, M2700N95, M2701N95, M2707N95, EZ23, EZ23S

DEAR MOLDEX USER:

Moldex produces disposable respirators with exhale valves that contain a natural rubber diaphragm. These products are very reliable but the shelf life can be affected by storage conditions such as heat and exposure to air and light. During a recent internal audit of retained products, we found that a limited number of exhale valves used in certain respirator models showed signs of hardening of the valve diaphragm. This condition may lead to reduced life of the valve and possible reduction in performance below the NIOSH standard. This was found in few manufactured lots that were 5 years or more past their date of manufacture.

Moldex has changed its shelf life policy to 4 years for these type products even though they are almost always sold and used in a time period less than four years.

Effective 4/8/2015, all models of Moldex Disposable Respirators with exhalation valves will be standardized to carry a **4 year shelf life** (herein referred to as expiration date). Most Moldex models with Exhale Valves already carry a 4 year expiration date. Therefore, standardization of the expiration date affects only those models that have previously carried an expiration date greater than 4 years.

Pursuant to this change in policy, Users must discontinue use of product that is more than 4 years past the date of manufacture. **These products are considered "User Qualified" products and applies only to models listed above.**

Products that are less than 4 years from date of manufacture can be used for up to four years from the manufacture date and do not require recall nor removal from service.

On all product with lot numbers between 110408-000 and 150407-999, Users must cross out existing expiration/use by year , subtract 5 years from the marked year and write in the amended expiration/use by year (e.g.2024 becomes 2019) . These are not considered "User Qualified" products and do not require return nor removal from service.



REVISION 1

April 14, 2015

DISPOSABLE RESPIRATOR WITH EXHALE VALVE

MOLDEX USER VOLUNTARY RECALL/STOP USE/RELABEL SHELF LIFE NOTICE

For Certain Moldex Model Numbers Produced before 4/8/2015

Series: 2300 N95, 2700 N95, EZ23 N95 and EZ23SN95

Model Numbers: 2236, 2300N95, 2300V, 2301N95, 2307N95, 2350N95, 2357N95, 2700N95, 2700V, 2701N95, 2707N95, M2700N95, M2701N95, M2707N95, EZ23, EZ23S

As part of Moldex's commitment to producing safe, quality and innovative products, we constantly monitor them to ensure the health and safety of our customers. During a recent internal audit of retained products, we found that a limited number of exhale valves used in the above respirators models showed signs of hardening of the diaphragm in the valve on some manufactured lots that were 5 years or more past their date of manufacture. This condition may lead to reduced life of the valve and possible reduction in performance below the NIOSH standard.

While not all masks exhibited hardening of the valve diaphragm, in an abundance of caution, we have decided to implement a voluntary recall and stop use of the above Moldex products that are older than 4 years from the manufacture date as marked on the product packaging.




As to **USERS**, the Voluntary recall requires that you not use product that is more than 4 years past the manufacturing date and that you return for replacement any product that **will not be used before 4 years from the manufacturing date** for replacement.

PRODUCTS SUBJECT TO THIS NOTICE are those Moldex models listed above that were produced prior to April 8, 2015. These products have manufacturing lot numbers preceding 150408-000.

Any Product held by you manufactured between April 8, 2012 and prior to April 8 2015 (this means lot numbers between 120408-000 and 150407-999) should be inspected by you and the shelf life expiration marking must be corrected on the master case and the individual boxes to reduce the shelf life expiration to 4 years from the manufacturing date. **Products that will be fully used within 4 years from date of manufacture already in your possession are safe for use and do not require recall nor removal from service.** Further guidance is provided in next section.



Attachment 1

Outer Package Labels

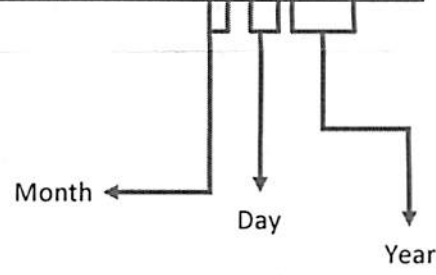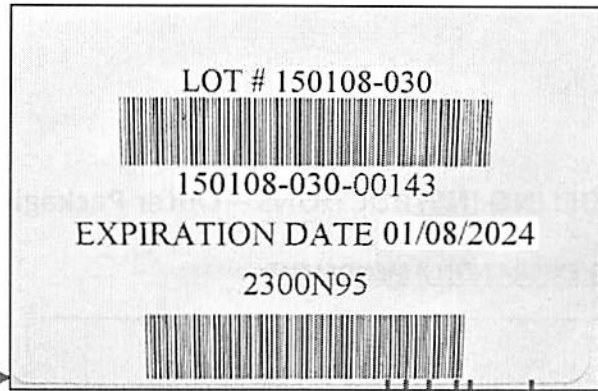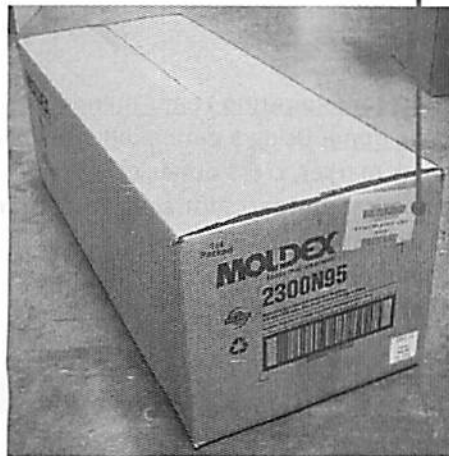


9 Year Expiration Date: (2015 + 9 years = 2024)

ManufacturingDate
Lot Number

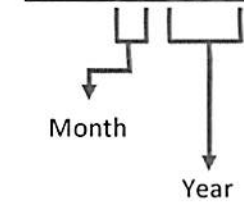
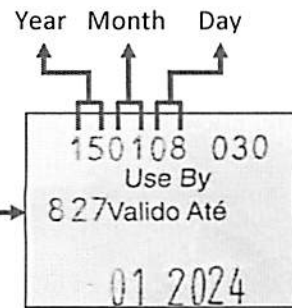


9 Year Expiration Date: (2015 + 9 years = 2024)

Attachment 2 - Inner Package labels

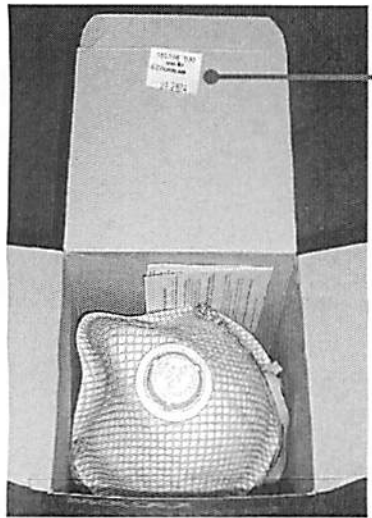


Manufacturing Date Lot

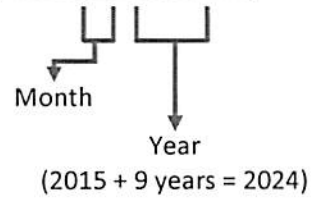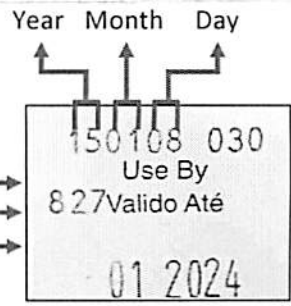

9 Year Expiration Date

4-YEAR USE BY YEAR AMENDMENT INSTRUCTIONS:

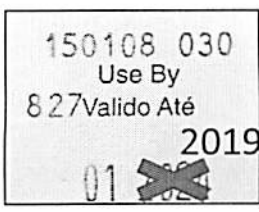

4 Year Use By Year Amendment to be done. Using a permanent dark black ink marker, cross out existing Use By year, subtract 5 years from expiration year, subtract 5 years from expiration year and write in the amended expiration year.(e.g. 2024 becomes 2019, etc.)

Tuesday, April 14th, 2015

**MOLDEX – DISPOSABLE RESPIRATOR WITH EXHALE VALVE –
Voluntary Recall/Stop Sale & Use/Re-label Shelf Life Notice - Activity Form**

Please Fill Out and e-mail to: customersupport@moldex.com or fax (310) 837-9563

Please check this box if you do not have the items listed below and return to Moldex per initial notice. No further action is required.

Check the appropriate box below: Moldex CSR: _____ Date: _____

Moldex Reseller: []	Moldex End-User: []
------------------------	------------------------

Item #	Amount in Stock (Total Cases, Boxes or Bags)	<input type="checkbox"/> We will re-label or mark each box, bag & case as instructed by Moldex. <i>(Lot # below)</i>	<input type="checkbox"/> We will request replacement product, please supply QCRA # and process replacement goods.
		PO#: QCLAB# _____	PO#: QCRA# _____
<input type="checkbox"/> 2300N95 M/L			
<input type="checkbox"/> 2301N95 Small			
<input type="checkbox"/> 2307N95 Low Profile			
<input type="checkbox"/> 2300V Vending Pack			
<input type="checkbox"/> 2350N95 Locker			
<input type="checkbox"/> 2357N95 Locker			
<input type="checkbox"/> 2236N95 Locker			
<input type="checkbox"/> 2700N95 M/L			
<input type="checkbox"/> 2701N95 Small			
<input type="checkbox"/> 2707N95 Low Profile			
<input type="checkbox"/> 2700V Vending Pack			
<input type="checkbox"/> M2700N95 M/L			
<input type="checkbox"/> M2701N95 Small			
<input type="checkbox"/> M2707N95 Low Profile			
<input type="checkbox"/> EZ 23 M/L			
<input type="checkbox"/> EZ 23S Small			

Contact Information: (please fill out one sheet per location) Dist. Account# _____ Ship Account #: _____

Company Name: _____ Contact Name: _____

Address: _____

City: _____ State: _____ Zip Code: _____ Country: _____

Phone: _____ Fax: _____ E-mail Address: _____

Note: If End-user, You purchased Moldex Products from: HD SUPPLY WHITE CAP

For Moldex Internal Use Only:

QCRA#: _____ Moldex Receiving: Quantity Received: _____ Tracking #: _____

Comments/Notes: _____